

AQUAMATE

NOVEL & MYSTICAL BEAUTY OF NEOGENESIS



Wireless Operation | Powerful Injection | Ergonomic Angle

Table of Contents

1. Intro.....	3
2. Strengths.....	6
3. Manual.....	15
4. Clinical Results.....	17
5. Composition.....	19
6. Specification.....	20
7. Related Products.....	21



“Compact” Wireless Device with Powerful Injection & Ergonomic Angle

Precise Control with Micro-Adjustable Volume, Dosage & Viscosity



Fast & Precise



Strong Suction



Auto- Adjustment



LCD Screen



Operation Time

9 Times Faster & More Precise than Manual Injection


You can perform Professional Mesotherapy Treatment with Ease

➤ Mesotherapy: minimally invasive skin treatments

Mesotherapy consists of a series of micro injections in the superficial layer of the skin of active ingredients that slowly diffuse into the underlying tissues.

Skin boosters are usually injected in the Dermis area of the skin tissue. To maximize the effect of the therapy, non skin-tearing method and treatment is crucial for the mesotherapy.

Minimally invasive, lower dose of drug compared to systemic treatment, **lower risk of systemic adverse events, simple application.**

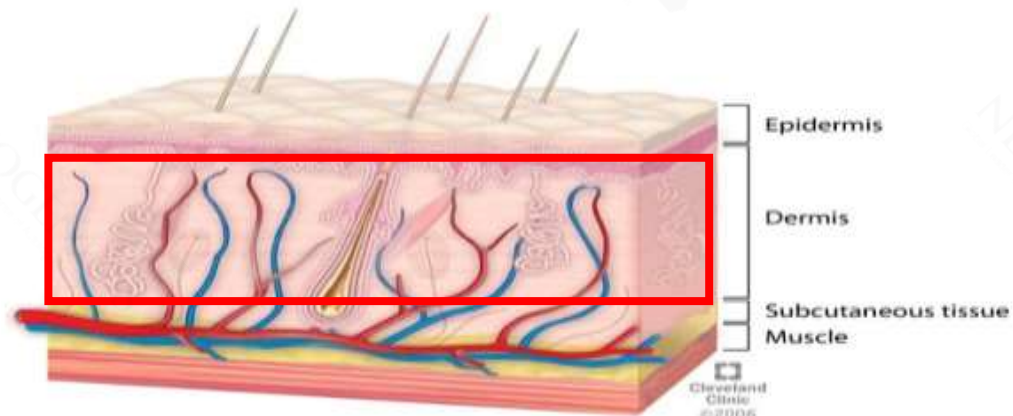

Manual Injection



- ✓ Manual Injection with needles requires a lot of effort and concentration.
- ✓ Very time-consuming accumulates fatigue in fingers and wrist.
- ✓ Uneven injection and inefficient.

AQUAMATE Injector




- ✓ For less fatigue and greater effectiveness, meso-gun is now pervasive in the market.
- ✓ 9 times faster treatment (34G 9-Pin)
- ✓ Even and Precise Injection

Suction, Injection & Anti-leakage Reverse-Flow


Distributes solution evenly through out the Dermis without Loss of Solutions

1 Contacts & Suctions Skin

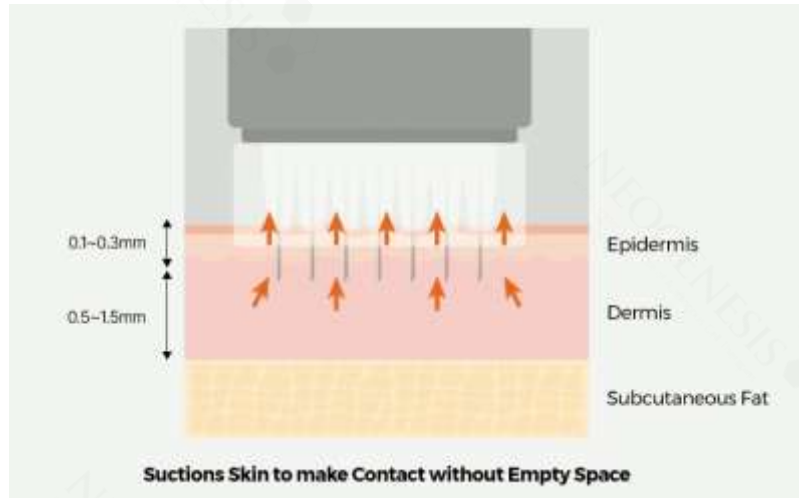


- ✓ Needles penetrate epidermis area.
- ✓ Suctions skin to make contact without any empty space.

2 Evenly distributed Injection

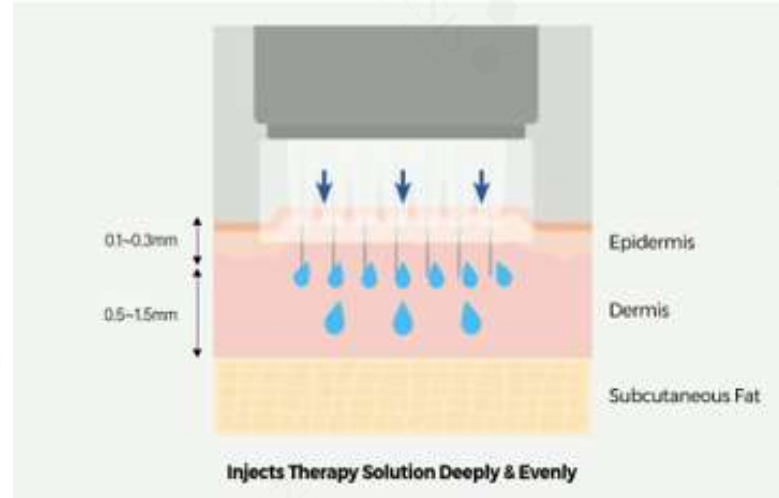

- ✓ AQUAMATE starts to injecting the solution into the dermis area.
- ✓ AQUAMATE needle deeply and evenly injects and spreads the solution.


3 Reverse-flow of Residual

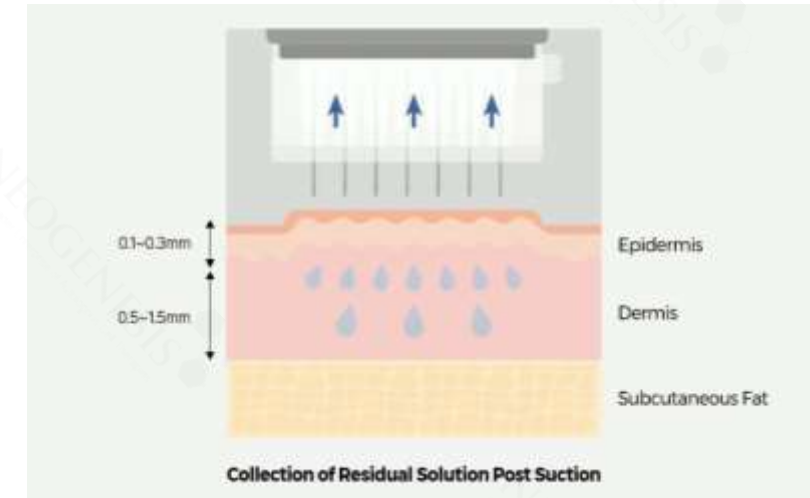

- ✓ AQUAMATE collects solution remaining on the needle after the injection.

Comprehensive All-in-one: Pump, Control Panel, & Injector

Aquamate is Even More Compact than our Previous Injector; LeFil

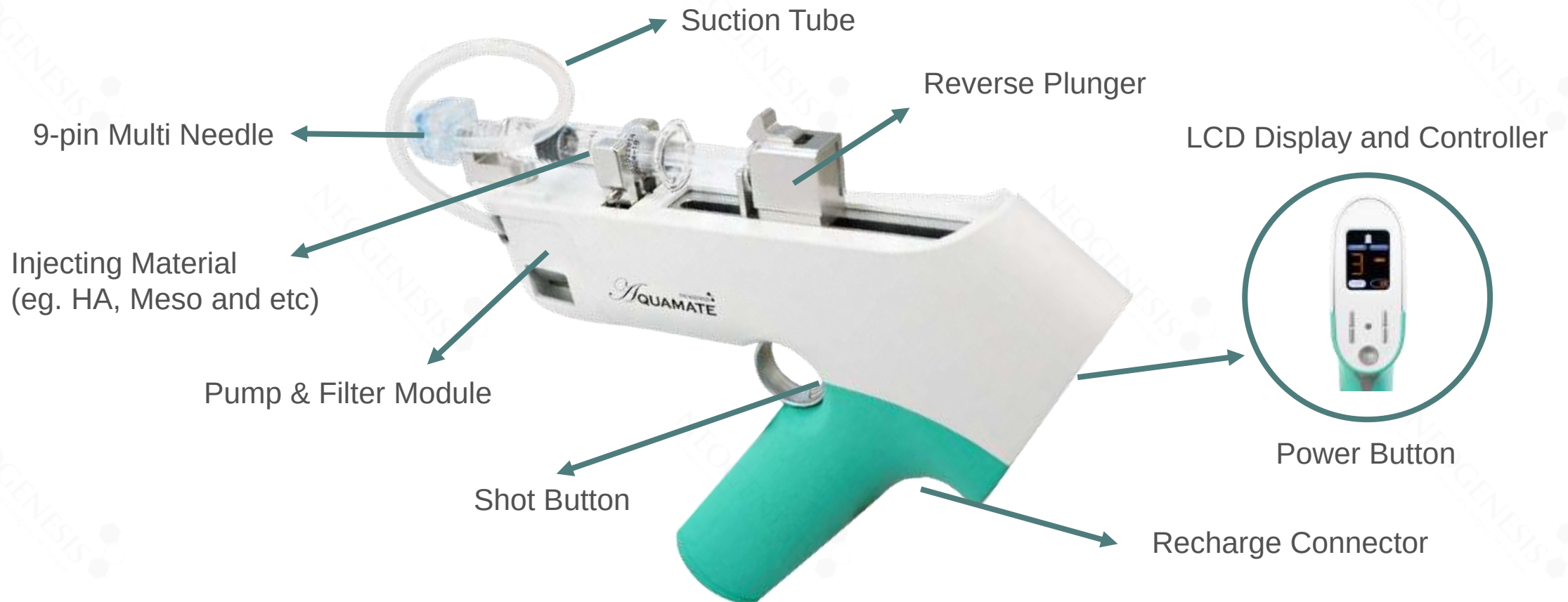

6 Advantages that make AQUAMATE the Best Mate

Compact, Wireless, Powerful Injection & Ergonomic Shooting Angle

Compact & Ergonomic Grip
Comfortable Handling

Leakage Prevention
Reverse Flow Plunger

Wide Compatibility
All Syringes (1/3/5 cc)



Powerful Injection
Strong Pump Module

Wireless Usage
90 Minutes of Operation Time

LCD Control Panel
Precise Micro-Control

48 Degree Injection Angle for Less Wrist Fatigue


The Optimal Angle is set without having to Aim the Injector Upright to the Skin



① Compact & Light Weighted

- ✓ Dimensions (W x D x H): 220 x 40 x 190mm
- ✓ Light body of 470g - easy on the wrist
- ✓ Optimized design through elimination of superfluous details for enhanced user-friendliness


② 48 Degree Handle to Reduces Fatigue

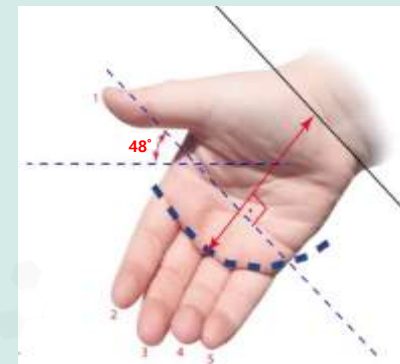

- ✓ Ergonomically designed handle(48 degrees) prevents fatigue from long term treatment.
- ✓ No need to hold the injector up right 90 degrees to the skin.

90 Minutes of Wireless Operation with One-time Charge

Rechargeable Lithium Batteries are Included with the Device

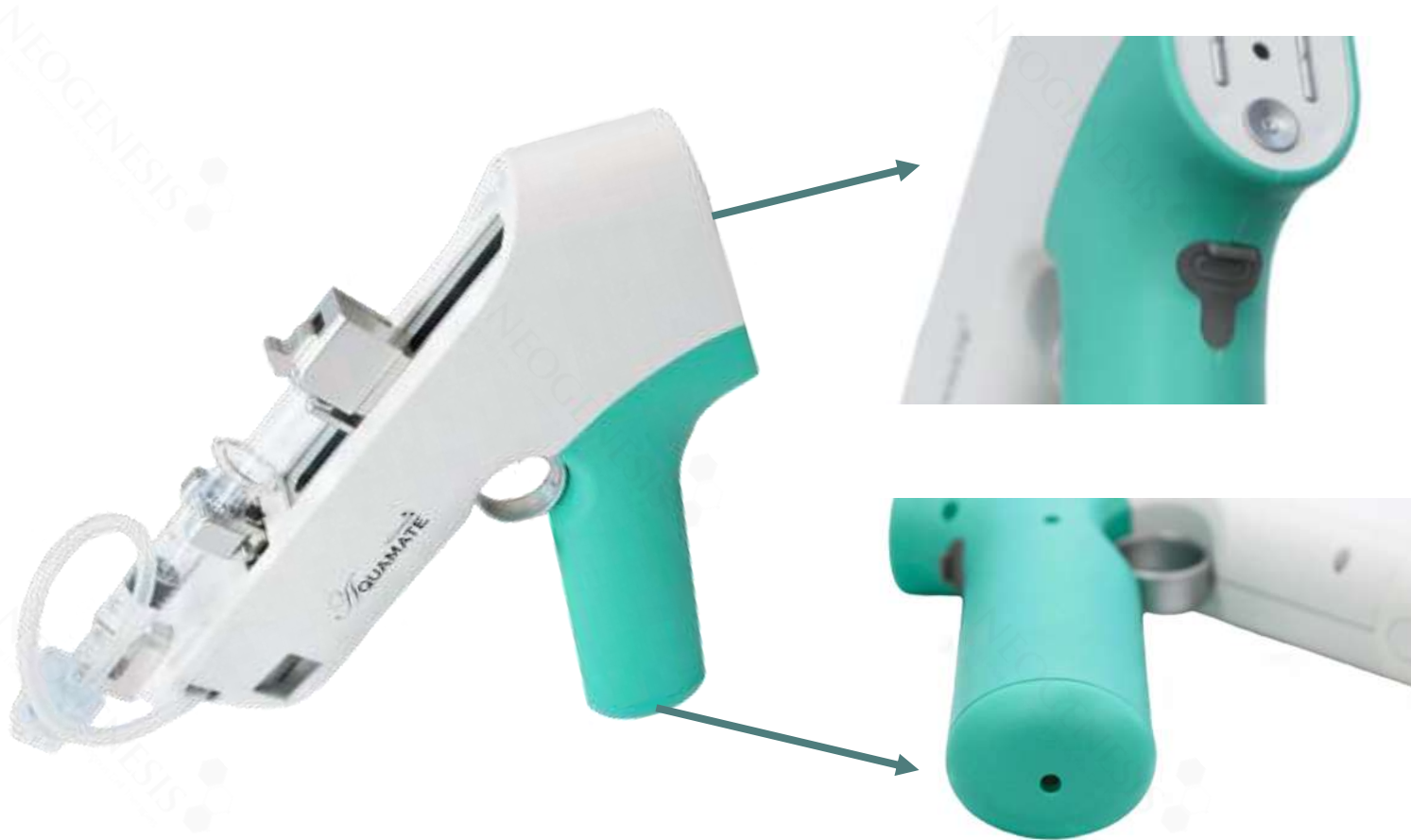

① Wireless Power

- ✓ Operates wireless for 90 minutes
- ✓ Can carry the device anywhere
- ✓ Power on without booting time
- ✓ May also be used wired


② Lithium Battery Equipped

- ✓ Rechargeable for long time use
- ✓ 1 Lithium battery included
- ✓ Full charge: 3 hours

Strong Suction is the Key to Preventing Solution Leakage

Maintains Injection Performance Semi-permanently with Replaceable Pump Filter

<The Mechanism of Module Pump>

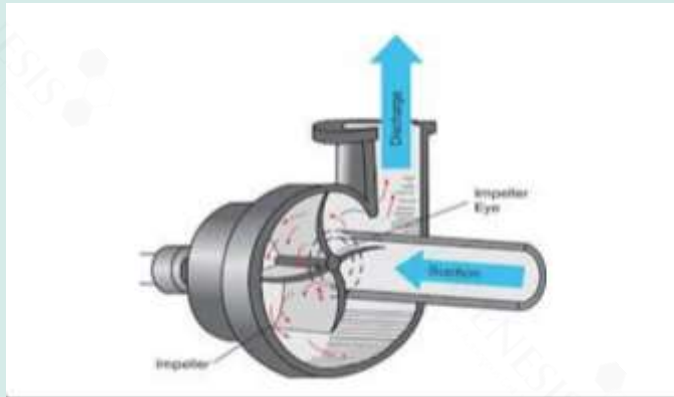

A pump moves fluid by creating a pressure difference between the inlet and outlet of the device. The movement of the piston, impeller, or other mechanisms in the pump creates this pressure difference and drives the fluid through the pump.

AQUAMATE ensures the best effect with a strong and stable pump. The pump's filter gauge can also be observed to indicate when to replace the pump.

① Strong pump module



- ✓ As the needles contact the skin, AQUAMATE suctions the skin leaving no empty space between the pin and the skin
- ✓ Strong pump module injects viscous solutions with injection resistance without leaking out from the skin

② Collection after each shot



- ✓ Pump reverses the plunger after each shot to collect any solution hanging out from the end of the pins to minimize any loss
- ✓ Pump also inhales any leaked solutions occurring from misuse through the suction tube to keep the needle clean and prepare for the next injection

Automatically adjusts the Plunger Power with a Sensor

Detects Pressure Generated during Injection and Operates in Line with the Settings

<The Mechanism of Pressure Sensor>

① Resistance Pressure Occurs



- ✓ The skin resists solution from entering the skin when it comes in contact with the pin.
- ✓ The injector's output is not the same as the settings you have set prior to the injection.

② Senses the Pressure



- ✓ A sensor is equipped in the injector's plunger to detect the amount of pressure occurring.
- ✓ The sensor then transmits the pressure gauge to the injector's program.

③ Adjusts Plunger Power



- ✓ The injector's program automatically adjust the distance that has to be covered by the plunger.
- ✓ The injector's output comes is in line with the original settings you have set prior to the injection.

1/3/5cc Syringes can All Be Used with AQUAMATE

Adjustable Syringe Dock and Reverse Plunger designed to Lock Syringes Down

① Width adjustable syringe dock



- ✓ Syringe Dock with Adjustable Width, Install various sizes of syringes (1/3/5 cc)
- ✓ With syringe dock, you can fasten or unfasten by twisting the dock to fit the syringe's width perfectly

② Plunger with a syringe lock



- ✓ With the syringe plunger you can fasten the syringe tightly, no discomfort during the procedure
- ✓ Adjustable plunger goes back and forth to fit the syringe perfectly


1.6 inch LCD Monitor with 4 Simple Control Buttons

Excellent Precision with Micro-Adjustable Volume, Dosage & Viscosity

LCD Monitor Indicators

- 
- ① Syringe Type: Select from 1/3/5cc syringe volume
 - ② Dosage per Shot: 1~9 level dosage setting
 - ③ Solution Viscosity: 1~5 level viscosity setting
 - ④ Pump Power: Displays pump mode on/off
 - ⑤ Battery State: Displays battery remaining

Control Buttons & Charge

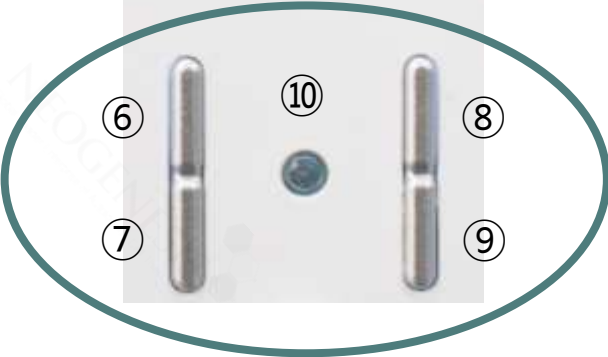- 
- ⑥ Left Upper Button: Viscosity Control
 - ⑦ Left Lower Button: Dosage Control
 - ⑧ Right Upper Button: Pump On/Off
 - ⑨ Right Lower Button: Syringe Selection
 - ⑩ Charge Indicator: Red Light
 - Blinking: Charging / Light On: Fully Charged




- ⑪ Bottom Round Button: Power On/Off
 - Press Shortly: Power On
 - Press for 2 Seconds and Let it go: Power Off


9 Individual Suction Channels Amplify the Suction Power

Ultra-thin 34G Needles Reduce Skin Damage and Speed Up Recovery

① Ultra Fine 34G 9-Pin

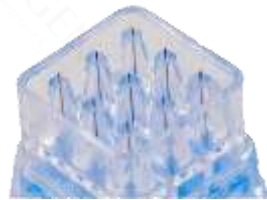


- ✓ 34 Gauge 9 multi pin (1mm fixed depth)
- ✓ State-of-the-art 3 step cutting for Ultra Fine Needles
- ✓ Uniform length of 100/5 tolerance

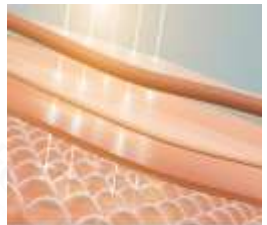

- ✓ Even distribution of drug through 9 pin needles
- ✓ Visible Embossing Effect (the shape of drug forming on skin) that represents how well injected

34G 9-pin

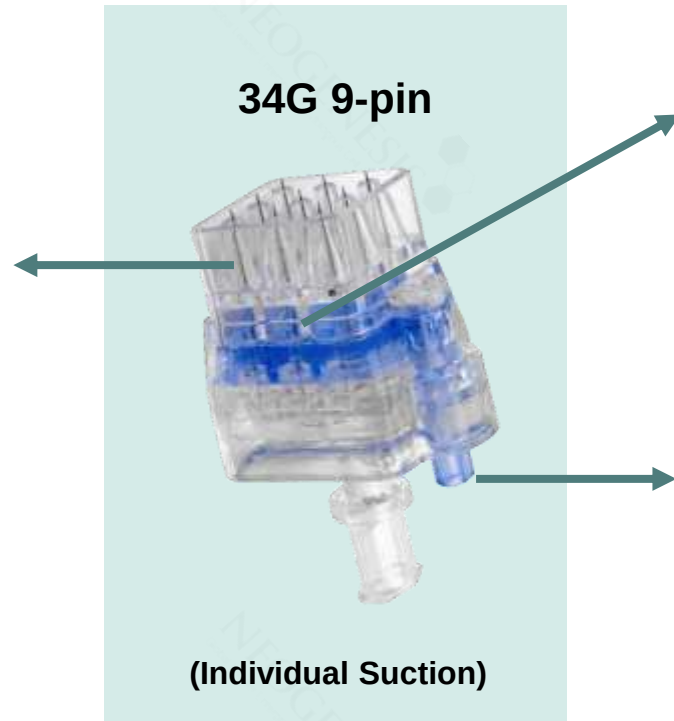

(Individual Suction)

② Individual Suction



- ✓ Suction channels are evenly distributed to 9 multi needles (Regular pin's suction applies on the entire surface of the pins)
- ✓ Suction power gets much stronger when passing through a narrower passage

③ Parallel Suction Inlet



- ✓ Suction inlet located in the same direction of procedure
- ✓ Makes easy to perform suction and secure the operator's view of treatment area during procedure

32G 9-pin with adjustable depth is also available to be sold separately.

AQUAMATE: The Best Mate for Mesotherapy


The Ideal Trinity of Compactness, Wireless, and Powerful Performance

Product Name	AQUAMATE	Brand 'D'	Brand 'G'	Brand 'A'
Product Image				
Size	220 x 40 x 190mm	200 x 45 x 200mm	180 x 50 x250mm	170 x 70 x 230mm
Weight	470g	495	513	475
34G 9 pin	O	X	X	X
Wireless Usage	O	X	X	X
Warranty	12 months + Lifetime Maintenance	12 months	X	12 months
Price	\$\$	\$\$\$\$\$	\$\$\$\$	\$\$

Install the product of Your Choice in 6 Simple Steps

You Can Follow the Visual Tutorial on NeoGenesis YouTube Channel

① Syringe Preparation

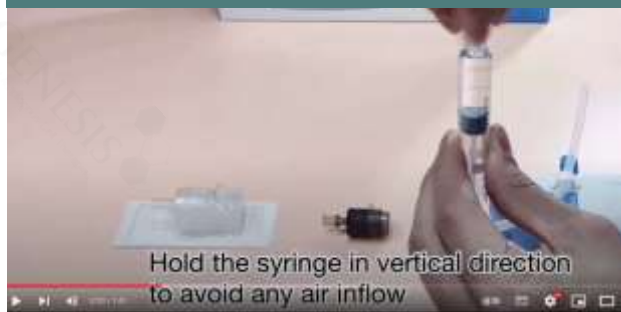

- ✓ Hold the syringe in vertical direction to avoid any air flow.

② Needle Installation



- ✓ Install 5 pin or 9 pin needle by turning it clockwise until it is firmly set.

③ Syringe Installation



- ✓ Install the syringe properly to the dented frame at each ends.

④ Fasten Syringe Dock



- ✓ Fasten the syringe dock so that the syringe does not move.

⑤ Lock Reverse Plunger



- ✓ Lock the plunger holder to prevent the hand-piece from popping out.

⑥ Adjust Settings



- ✓ Modes: Select Size, Viscosity, Dosage, Pump On/Off

Find the Right Settings based on the Solution Type

Adjust according to the Number of Shots needed for each Specific Treatment

Dosage \ Volume of Solution	1cc	3cc	5cc
1	80 shots	240 shots	320 shots
2	40 shots	120 shots	160 shots
3 (recommended)	20 shots	60 shots	80 shots
4 (recommended)	16 shots	48 shots	64 shots
5 (recommended)	14 shots	42 shots	56 shots
6	12 shots	36 shots	48 shots
7	10 shots	30 shots	40 shots
8	8 shots	24 shots	32 shots
9	Manual Dosage Mode: Press and hold the shot button until desired amount of solution is injected.		

Above settings are based on REGENOVUE AQUASHINE with viscosity at level 2 and syringe at 3cc.

- ✓ REGENOVUE AQUASHINE is a HA skin booster with 3cc solution in a 5cc syringe.

It is Better Not to Use AQUAMATE while Charging

The Batteries will last longer and the Injection Power is More Stable after Full Charge



- AQUAMATE is sold exclusively for medical use only. Public general sales are prohibited and is only for qualified physicians.
- Check the filter condition regularly and replace it when the filter gauge indication exceeds 50%.
- Check the silicone suction tube for foreign substances and keep it clean and dry.
- Check the needles' and syringes' sealing condition. Do not use if there is damage to the packaging of the products.
- Separate the used disposable needle and syringe from the main body and dispose safely.
- Do not exert excessive pressure when using the device.
- **Do not use Fast-charge adaptor. Use Adaptor with output of below 2A.**
- Separate the adaptor from the machine and organize properly after full charge.
- Store the product in a well ventilated and dry place at room temperature.



Proven Performance with Minimal Damage & Leakage

Customers Praised the Stability of Injection Even After Consecutive & Repeated Use



Candidate Detail	Female, 32
Area of Concern	Dark Spots
Used Pin	34G 9-pin
Number of Sessions	3



Candidate Detail	Female, 22
Area of Concern	Acne
Used Tip	32G 9-Pin (Depth Adjustable)
Number of Sessions	4

Dosage 3 and 4 are recommended for most skin boosters.

An Injector Body, 1 Extra Pump Filter and an Adaptor

1 Piece each of 34G 9-Pin & 32G 9-Pin are Included with the Purchase




1ea Full Device, 1ea Extra Pump Filter, 2ea Extra Suction Tubes,
1ea Adaptor, 1ea USB-C type Cable, 1ea User Manual, **1ea 34G 9-pin, 1ea 32G 9-pin**

9-Pins can be sold separately in units of 1 piece.



(20EA/box)

**34G 9-Pin
(Individual Suction)**

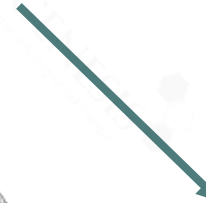


(10EA/box)

**32G 9-Pin
(Depth Adjustable)**

Guaranteed Quality from GMP & ISO Certified Facilities

Complies with International Standards for Electrical Devices

Specification of AQUAMATE	
Compatibility	All skin boosters and mesotherapy syringes
Power Supply	100~240V / 50~60 Hz. 0.35A
Rated Output	5.0V = 2.0A
Battery Type & Rating	Rechargeable lithium-ion 3,500 mAh 3.7V x 2EA = 7.4V DC
Dimensions (W x D x H)	220 x 40 x 190mm
Weight	470g
Consumables	34G 9-pin needle (individual suction) 32G 9-pin needle (depth adjustable)
Control Panel	Graphic LCD 1.6 inch Screen

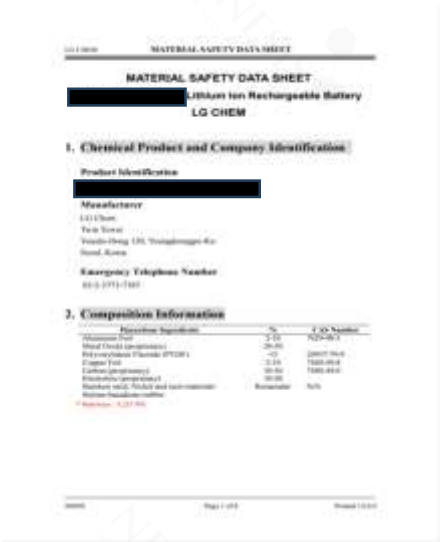

Get Ready with Neo-Cain Anesthetic Cream

Boost Recovery with BETA MASK's Soothing, Cooling & Anti-Inflammation Effects



Neo-Cain Cream

High-quality anesthetic cream with 10.56% lidocaine

- ✓ Quick & powerful effects
- ✓ Made in strict GMP facilities in South Korea
- ✓ Competitive price



Beta Mask

Experience professional post-treatment care

- ✓ Cell regeneration
- ✓ Excellent aftercare
- ✓ Outstanding recovery effects



Optimized for REGENOVUE Skinbooster Series

High-Purity HA for Glowing Skin & PN for Skin Regeneration with Anti-Wrinkle Effects



Regenovue Aquashine

Non Cross-linked Skin Booster

- ✓ Quick & powerful effects
- ✓ CPNP Certified
- ✓ 3~5 months duration



Regenovue Aquashine Plus

Partially Cross-linked, long lasting

- ✓ Strong Effect
- ✓ CPNP Certified
- ✓ 6~8 months duration



Regenovue PN

Non Cross-linked Skin Booster

- ✓ PDRN and HA Skinbooster
- ✓ CNPN Certified
- ✓ 3~5 Months duration

*A*QUAMATE

THE PERFECT MATE FOR MESOTHERAPY

Address: #307 Think Factory Bldg. C, 150, Yeongdeungpo-ro
Yeongdeungpo-gu, Seoul, South Korea (07292)
Tel: +82-2-583-1348 **Fax:** +82-2-2038-0848
Email: info@neogenesis.co.kr
Website: www.neogenesis.co.kr